

APPLYING MEN'S LIBERTY




1

Keep foreskin back completely throughout application. Prepare the skin with the BioPlus+ no sting barrier wipe.



2

Attach the faceplate to the tip of the penis with the center opening directly in line with the urinary opening. Smooth down petals.

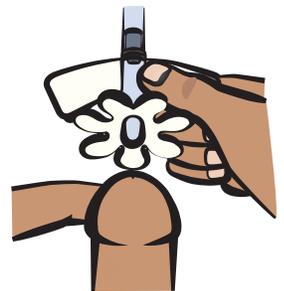

3

Apply the faceplate seal around the head of the penis flat and wrinkle free. Be sure the hydrocolloid is pressed into all the ridges and grooves in the skin.



4

To Intermittent Cath: Remove the plug from the port by holding plastic housing firmly in one hand and rolling the black plug upwards towards your stomach. Lubricate and insert intermittent catheter through the device. Cath as normal. Clean out excess lube with Q-tip. Reinsert plug once complete.

YOUR FIRST FEW DAYS

As a result of using pads or condom catheters, your skin currently has **A LOT** of excess moisture. Fortunately for you, Men's Liberty just **LOVES** moisture, so it will soak it all up, until your skin returns to a normal moisture balance. However, that means that it takes up to three consecutive applications of Men's Liberty for you to get to a full 24 hour wear time.



Your first Men's Liberty application should last 3-6 hours
(hint: apply first thing in the morning)



Your second Men's Liberty application should last 8-12 hours
(hint: replace around lunchtime)



Your third Men's Liberty application should last 18-24 hours
(hint: replace before bed on day 1)

Follow these hints and by day 2 you should be up to a regular 24 hour wear time.

Always wash your anatomy with a non-moisturizing bar soap such as Ivory or Gold Bar Dial. **DO NOT** use liquid soaps as they contain moisturizers which will reduce adhesion.

Make sure your anatomy is **COMPLETELY DRY** before applying Men's Liberty.

DO NOT ping-pong between Men's Liberty and other products like pads or condom catheters. This will shorten your wear time!

